




StatSeal®


Hemostatic Sealant

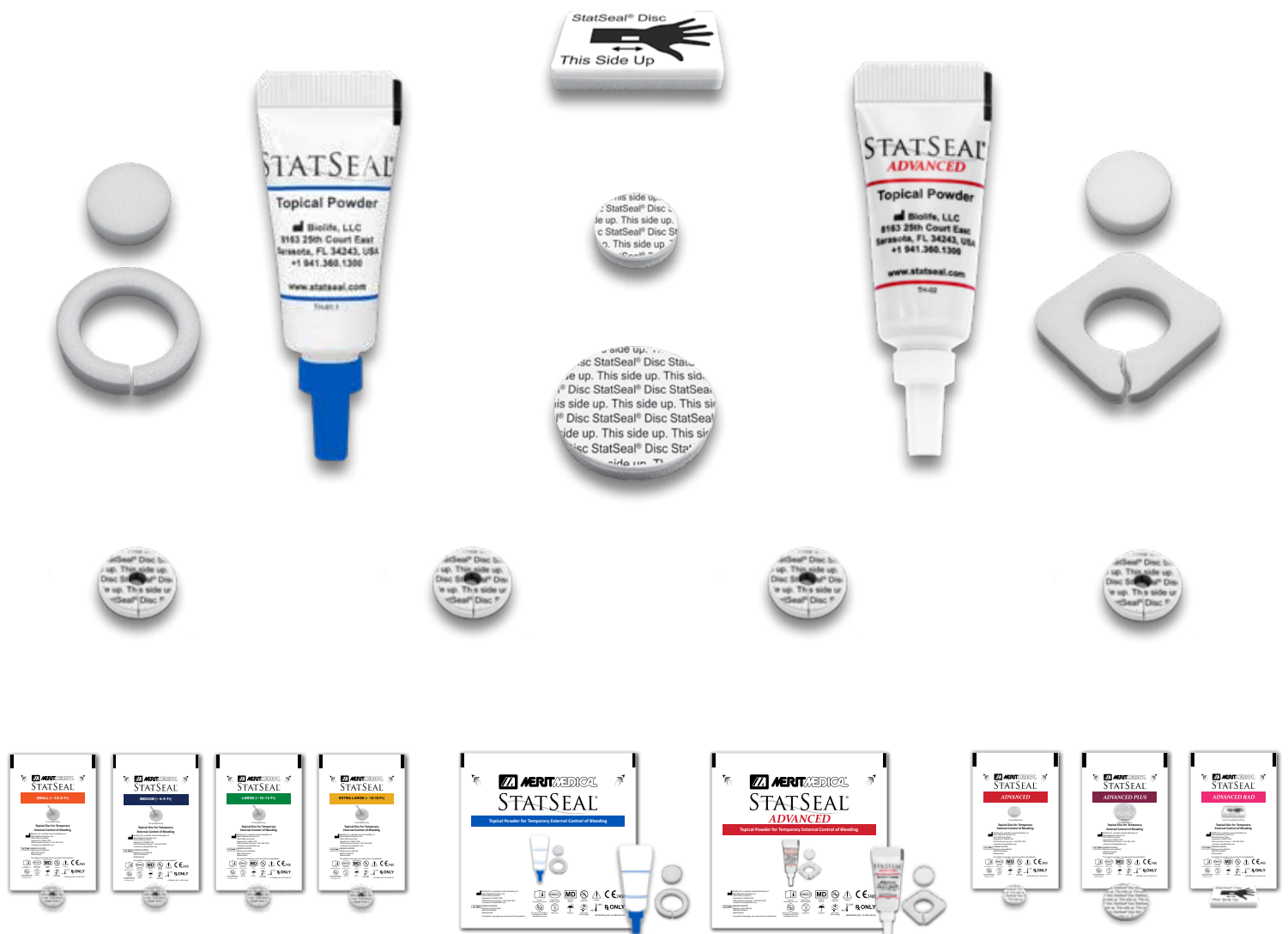

LEADER IN HEMOSTASIS

StatSeal Hemostatic Sealant

Built for every bleed.

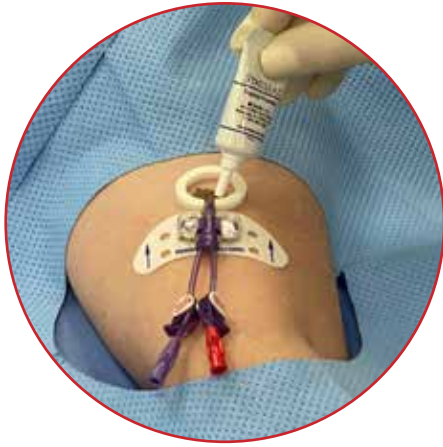

Merit Medical OEM is excited to introduce StatSeal, a compact, kit-ready solution designed to support rapid external bleeding control. StatSeal's unique formula quickly forms an occlusive seal to stop bleeding and oozing, helps streamline post-procedure care, and eases clinician burden, saving both time and money for clinicians and improving outcomes for all patients.^{1,2}

What a Hemostatic Sealant Does

StatSeal dehydrates and binds blood proteins, using a hydrophilic polymer and potassium ferrate, creating a durable seal without relying on the clotting cascade. It creates a low-pH environment that protects the site while natural healing continues beneath.¹

Dramatically Reduce Rebleeds and Recovery Time with StatSeal

Two global randomized controlled trials with 2500+ patients


94%
REDUCTION
IN REBLEEDS^{3,4}


50%
REDUCTION IN
TIME TO HEMOSTASIS^{3,4}


87%
REDUCTION IN
DISCHARGE DELAYS³

Seamless integration. Broad procedural utility. Available in both powder and disc form and a wide range of sizes to support diverse clinical applications.

FEATURES & BENEFITS

Rapid Hemostasis

- Quickly forms an occlusive seal to stop bleeding & oozing
- Works independent of clotting cascade, making it effective for patients on anticoagulation therapies^{3,4}
- Simplifies post-procedure care & eases clinician burden¹⁻⁴

Ideal For:

- Sheath & catheter removals
- Indwelling catheters
- Any procedure causing external bleeding

Proven, Safe Technology

- Hydrophilic polymer & potassium ferrate blend
- Hypoallergenic with no age contraindications
- Effective for patients prone to bleeding^{3,4}

Kit-Ready

- Easy-open foil pouches
- Broad size range to meet procedural needs
- Seamless pack and tray integration

RELIABLE

Keeps access/catheter sites dry and intact until scheduled dressing changes.^{1,2}

STATSEAL

RAPID

Forms a seal over the site to prevent bleeding and oozing.⁵

STATSEAL

REPRODUCIBLE

Works even for anticoagulated patients, ensuring universal consistency.^{3,4,6}


ORDERING INFORMATION

Catalog Number	Product Description	Size
DA628	StatSeal Advanced disc	1.4 cm
DAP629	StatSeal Advanced Plus disc	2.2 cm
DAR630	StatSeal Advanced RAD disc	2.4 cm x 1.2 cm
DS624	StatSeal Small disc	2.6F – 6F
DM625	StatSeal Medium disc	6F – 9F
DL626	StatSeal Large disc	10F – 14F
DX627	StatSeal Extra-large disc	15F – 18F
LP636	StatSeal Powder	Standard Volume
LP637	StatSeal Advanced Powder	Double Volume


CUSTOMIZATION

Custom packaging and private labeling available upon request.

ORDERING INFORMATION

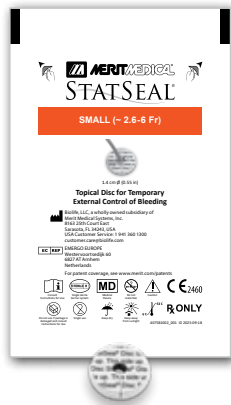


StatSeal Small disc
DS624

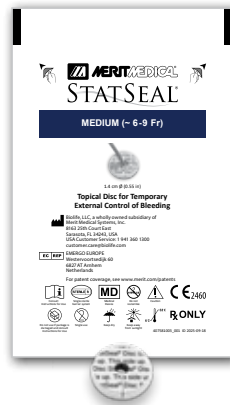

StatSeal Medium disc
DM625



StatSeal Large disc
DL626

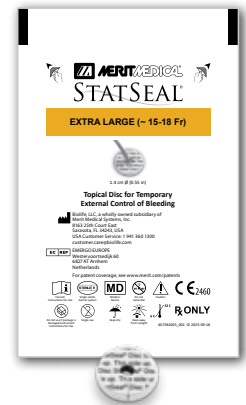


StatSeal Extra Large disc
DX627




StatSeal Powder
LP636




StatSeal Advanced powder
LP637

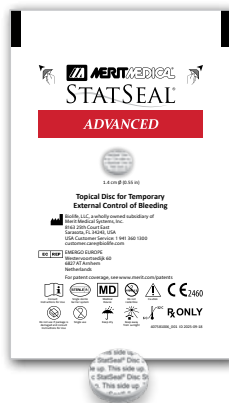

StatSeal Advanced disc
DA628

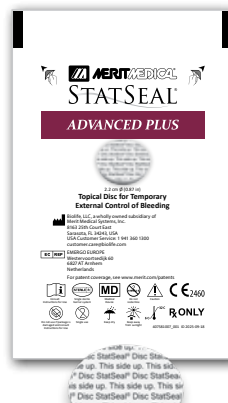

StatSeal Advanced Plus disc
DAP629

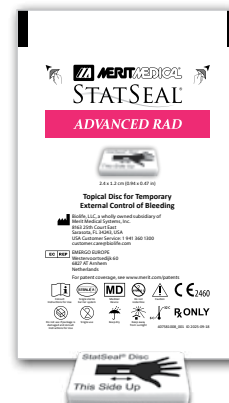

StatSeal Advanced RAD disc
DAR630

Built for Every Bleed

Insertion-Site Care

StatSeal helps hospitals protect patients and ease clinician burden. Its patented hypoallergenic formula minimizes dressing disruptions.^{1,2}

- PICC/CVC insertions
- Dialysis catheters
- Peripheral arterial catheters
- Umbilical catheters
- ECMO cannulas
- Temporary hemodynamic support devices
- Drainage catheters
- Tracheostomies
- PEG tubes

Closure

StatSeal accelerates hemostasis and standardizes hospital procedures, shortening recovery, enabling same-day discharge, and reducing complications across a range of high-volume procedures.^{3,4,6,7}

- Transradial and transfemoral access
- Fistuloplasty
- Cardiac ablation
- Vascular closure

Connect with our Merit OEM product development team to get started: www.meritoem.com

REFERENCES

1. Hastings and Barton. 2024. "Rapid Haemostasis to Achieve Dressing Longevity: Evaluation Trial Results Using StatSeal Catheter Exit Site Protection." *Br J Nurs* 33, no. 14:S8–S14. doi: 10.12968/bjon.2024.0164.
2. Blough, Hinson, and Hen. 2010. "The Science of a Seal for PICC Line Management: Bio Seal CVC Powder." *J VAS Access* 15, no. 2: 66–73.
3. Proscia et al. 2025. "A Randomised Control Trial to Compare Conventional and Haemostatic Dressings in Radial Arterial Access: Assessment of Radial Artery Complications Whilst Achieving Rapid Haemostasis (ARCH Trial)." *Int J Cardiol* 420: 132740. doi: 10.1016/j.ijcard.2024.132740.
4. Safirstein et al. 2022. "Radial Hemostasis Is Facilitated with a Potassium Ferrate Hemostatic Patch: The STAT2 Trial." *JACC Cardiovasc Interv* 15, no. 8: 810–19. doi:10.1016/j.jcin.2021.12.030.
5. Data on File.
6. Bagur et al. 2022. "Postprocedural Radial Artery Compression Time in Chronic Anticoagulated Patients Using StatSeal: The PRACTICAL-SEAL Study." *Int J Cardiol* 346:14–17. doi: 10.1016/j.ijcard.2021.11.011.
7. Ayyaz et al. 2019. "Accelerated Patent Hemostasis Using a Procoagulant Disk: A Protocol Designed to Minimize the Risk of Radial Artery Occlusion Following Cardiac Catheterization." *Cardiovasc Revasc Med* 20, no. 2: 137–42. doi: 10.1016/j.carrev.2018.03.026

Before using, refer to Instructions for Use for indications, contraindications, warnings, precautions, and directions for use.



COMPONENTS | DEVICES | POSSIBILITIES™

Merit Medical Systems, Inc. (NASDAQ: MMSI) has been designing, manufacturing and marketing single-use medical devices to customers worldwide for over 25 years. Throughout its evolution to a successful global company, Merit has developed the organizational infrastructure required to succeed in a competitive, dynamic and highly regulated industry. Merit's OEM customers benefit not just from Merit's components and devices, but also from the organization that supports them.

